

## **Upgrading Your Base Station Firmware**

- 1. Connect the Delsys Base Station to the PC.
- Download the "Delsys Software Update Tool" from <u>https://www.delsys.com/</u>. Please go to Support>Software Downloads, and scroll to find the "Delsys Software Update Tool". Install the software to your computer.
- 3. Open the Delsys Software Update Tool.
- 4. If the Trigno Base Station section has a yellow 'Caution' triangle, the connected base station's current firmware is not compatible with the installed version of EMGworks.





5. Click 'Update...' in the Trigno Base Station section. The base station will immediately begin to update.

Delsys Software Update Tool v1.7     Connect any Delsy     Review a summary of the u	2.0 s hardv odates ap	ware to the PC to determine if a firmware upd	- C × late is required. <u>Need to downgrade?</u>
EMGworks	0	Installed version 4.7.1	
Trigno Base Station Reset Base Flash Memory	Δ	Serial: SP-W02C-1039 (BID-2131) Manufacture date: Friday, April 13, 2018 Firmware: MA2906-BE1502-DS0801-US2003-DA0900	Update
Trigno Sensors	0	Ready to pair.	Pair/Query sensor
		Note: Bluetooth compatible sensors must be updated via our Firmware Update app on the Google Play store	
User Guide (MAN-029)			
El	MGworks I	icense Key: KJMB-W59D-XJHL-7PZF-4SMJ-TXTZ-8JHL-F7CS	

Connect any Delsy Review a summary of the u	rs hardv pdates app	ware to the PC to determine if a firmware upda	ate is require <u>Nee</u>	d. to downs	grade?
EMGworks	0	Installed version 4.7.1			
Trigno Base Station Reset Base Flash Memory	Δ	Serial: SP-W02C-1039 (BID-2131) Manufacture date: Friday, April 13, 2018 Firmware: MA2906-BE1502-DS0801-US2003-DA0900			
Trigno Sensors	0	Ready to pair. Note: Bluetooth compatible sensors must be updated via our Firmware Update app on the Google Play store	Pair/Query	sensor	
Updating (1 of 4)	5				

TEL: 508-545-8200 FAX: 508-975-4551



6. The base station will now be updated to the most current version of firmware. Check to make sure there is a green check in the Base Station section.

Delsys Software Update Tool v1.7.2     Connect any Delsys	a hardware to the PC to determine if a t	− □ ×
Review a summary of the up	dates applied in this session	Need to downgrade?
EMGworks	Installed version 4.7.1	
Trigno Base Station Reset Base Flash Memory	Serial: SP-W02C-1039 (BID-2131) Manufacuture date: Friday, April 13, 2018 Firmware: MA2909-BE1503-DS0801-US20	04-DA0900
Trigno Sensors	Ready to pair. Note: Bluetooth compatible sensors must be via our Firmware Update app on the Google	Pair/Query sensor e updated e Play store
User Guide (MAN-029) EM	Gworks License Key; KJMB-W59D-XJHL-7PZF-4SMJ-	TXTZ-8JHL-F7CS



## **Downgrading Your Base Station Firmware**

- 1. Connect the Delsys Base Station to the PC.
- Download the "Delsys Software Update Tool" from <u>https://www.delsys.com/</u>. Please go to Support>Software Downloads, and scroll to find the "Delsys Software Update Tool". Install the software to your computer.
- 3. Open the Delsys Software Update Tool. Please refer to the "Delsys Trigno Legacy Sensor Firmware Guide" to determine the firmware version needed for the EMGworks version installed.
- 4. Click on "Need to downgrade?" at the top right.

Connect any Delsy Review a summary of the u	rs hardv	vare to the PC to determine if a firmware up	date is required. Need to downgrad
EMGworks Trigno Base Station	0	Installed version 4.5.4 Serial: SP-W02C-1039 (BID-2131) Manufacuture date: Friday, April 13, 2018 Firmware: MA2909-BE1503-DS0801-US2004-DA0900	
Trigno Sensors	0	Ready to pair. Note: Bluetooth compatible sensors must be updated via our Rimware Update app on the Google Play store	Pair/Query sensor
r Guide (MAN-029)			



5. Select the appropriate EMGworks version from the drop down menu and click "Downgrade Base". The base station will immediately begin to downgrade.

Review a summary of th	isys nardware to the PC to determine	e if a firmw	are	pdate	Need to downgrad
EMGworks	Installed version 4.5.4				
Trigno Base Station	@ Downgrade	-		×	1
Reset Base Flash Memory	Select downgrade option:				
Trigno Sensors	EMGworks 4.5.4   MA2906-BE1502-DS0801-	US2003-DA090	0	~	Pair/Query sensor
	EMGworks 4.3.2   MA2903-BE1500-DS0801- EMGworks 4.5.4   MA2906-BE1502-DS0801-	JS2001-DA090 US2003-DA090	)0 )0		
	Current Base Firmware: MA2909-BE1503-DS0801-US2004-DA0900	Downg	grade (	Base	
	Sensor Firmware:	Downg	arade \$	Sensor	
	F ALL VIEW	Contraction of the local division of the loc	-	and the second sec	
	Pair to view				
r Guide (MAN-029)			_		
<del>er Guide (MAN-029)</del> Delsys Software Update Tool	E v1.7.2.0			L.	
er Guide (MAN-029) Delsys Software Update Tool Connect any De Review a summary of t	E v1.7.2.0 elsys hardware to the PC to determin @ Downgrade	e if a firmw — 🗆	vare	update ×	e is required. Need to downgrad
er Guide (MAN-029) Delsys Software Update Tool Connect any De Review a summary of t EMGworks	E v1.7.2.0 elsys hardware to the PC to determin © Downgrade Select downgrade option:	e if a firmw — 🗆	rare	update ×	e is required. Need to downgrad
er Guide (MAN-029) Delsys Software Update Tool Connect any De Review a summary of t EMGworks	E V1.7.2.0 Elsys hardware to the PC to determin Output Downgrade Select downgrade option: EMGworks 4.5.4   MA2906-BE1502-DS0801-US	e if a firmw — 🗆 2003-DA0900	vare	update ×	e is required.
er Guide (MAN-029) Delsys Software Update Tool Connect any De Review a summary of t EMGworks Trigno Base Station	E E V1.7.2.0 Elsys hardware to the PC to determin C Downgrade Select downgrade option: EMGworks 4.5.4   MA2906-BE1502-DS0801-US: Current Base Firmware:	e if a firmw — □ 2003-DA0900	vare	update ×	e is required. Need to downgrad
er Guide (MAN-029) Delsys Software Update Tool Connect any De Review a summary of t EMGworks Trigno Base Station Reset Base Flash Memory	E  V1.7.2.0  elsys hardware to the PC to determin  Downgrade Select downgrade option: EMGworks 4.5.4   MA2906-BE1502-DS0801-US2  Current Base Firmware: MA2909-BE1503-DS0801-US2004-DA0900	e if a firmw – 2003-DA0900 Downgrad	vare i	update	e is required. Need to downgrad
er Guide (MAN-029) Delsys Software Update Tool Connect any De Review a summary of t EMGworks Trigno Base Station Reset Base Flash Memory Trigno Sensors	E E V1.7.2.0 Elsys hardware to the PC to determin Downgrade Select downgrade option: EMGworks 4.5.4   MA2906-BE1502-DS0801-US2 Current Base Firmware: MA2909-BE1503-DS0801-US2004-DA0900	e if a firmw – 2003-DA0900 Downgrae	vare (	update	e is required. Need to downgrad
er Guide (MAN-029) Delsys Software Update Tool Connect any De Review a summary of t EMGworks Trigno Base Station Reset Base Flash Memory Trigno Sensors		e if a firmw – 2003-DA0900 Downgrav	vare (	update ×	e is required. Need to downgrad
er Guide (MAN-029) Delsys Software Update Tool Connect any De Review a summary of t EMGworks Trigno Base Station Reset Base Flash Memory Trigno Sensors	E  V1.7.2.0  elsys hardware to the PC to determin  C Downgrade Select downgrade option: EMGworks 4.5.4   MA2906-BE1502-DS0801-US2  Current Base Firmware: MA2909-BE1503-DS0801-US2004-DA0900  Sensor Firmware: Pair to view	e if a firmw – 2003-DA0900 Downgras	de Bas	update ×	e is required. Need to downgrad
er Guide (MAN-029) Delsys Software Update Tool Connect any De Review a summary of t EMGworks Trigno Base Station Reset Base Flash Memory Trigno Sensors	Fail to view E v1.7.2.0 Elsys hardware to the PC to determine Image: Downgrade Select downgrade option: EMGworks 4.5.4   MA2906-BE1502-DS0801-US2 Current Base Firmware: MA2909-BE1503-DS0801-US2004-DA0900 Sensor Firmware: Pair to view	e if a firmw – 2003-DA0900 Downgras Downgras	de Bas	update	e is required. Need to downgrav



<ul> <li><u>Review a summary o</u></li> </ul>	Downgrade			×	Need to down
EMGworks	Select downgrade option:			_	
	EMGworks 4.5.4   MA2906-BE1502-DS0801-U	S2003-DA0	0900	~	
Trigno Base Statio Reset Base Rash Memor	Current Base Firmware: MA2906-BE1502-DS0801-US2003-DA0900	Not	Available	,	
Trigno Sensors					Pair/Query sensor
	Sensor Firmware: Pairto view	Dov	wngrade (	Sensor	
	Downgrading (1 of 4)				

6. Once the base station has finished downgrading, you may close out of the Software Update Tool.